



## **Premier Guard USA Fact Sheet for Healthcare Personnel under the Emergency Use of Authorized Disposable, Single-Use Surgical Masks During the COVID-19 Pandemic**

The Premier Guard USA (“PGUSA”) Surgical Face Mask is a disposable, single-use surgical mask consisting of 3-ply polypropylene materials, a nose wire, and elastic ear loops that does not contain natural rubber latex, any drugs, biologics, nanoparticles, or antimicrobial/antiviral agents.

The product is not intended to replace the need for FDA-cleared surgical masks or FDA-cleared or authorized respirators.

In addition, the PGUSA Surgical Face Mask is not intended to provide protection against pathogenic biological airborne particulates and is not recommended for use in aerosol generating procedures and any clinical conditions where there is significant risk of infection through inhalation exposure; and

The PGUSA product has not been FDA cleared or approved.

Rather, it has been authorized by the FDA under an EUA for use in healthcare settings by HCP as PPE to provide a physical barrier to fluids and particulate materials to prevent HCP exposure to respiratory droplets and large particles during surgical mask shortages resulting from the COVID-19 pandemic

This product is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of medical devices, including alternative products used as medical devices, during the COVID-19 outbreak, under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.

# PREMIER | GUARD USA

**REF** 20-1002-SFM3

## Premier Guard USA Surgical Face Mask

### ASTM LEVEL 3

Disposable surgical mask

Elastic ear loops do not contain natural rubber latex

3-PLY Polypropylene Materials


Dimensions 17.5 cm x 9.5 cm

Blue, Non-Sterile and Single-Use


**Composition:** Direct body contact areas may include the face, mouth, nose, and ears.

Layer	Material	Body Contact (intact skin)
Inner Layer	Non-woven Polypropylene	Direct
Middle Layer	Melt Blown Polypropylene	Indirect
Outer layer	Non-woven Polypropylene	Indirect
Nose Strip	Polyethylene laminated double sided 24-gauge soft annealed carbon steel wire tie	Indirect
Elastic ear loops	Polyester/Spandex	Direct

### Instructions for Use:

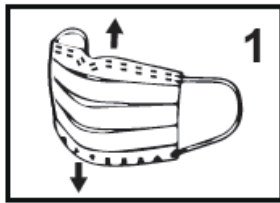

1. Spread out the mask and pull it apart in the up and down direction

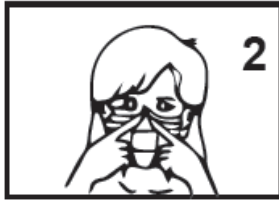

2. Fix the nose clip in the shape of nose



3. Pull the mask to the chin

**Replace mask after a single use or when wet or soiled. Do not reuse.**

Premier Guard USA LLC  
460 Briarwood Drive  
Jackson, MS 39206

Questions:  
888-706-1231 (9am-5pm CST)  
[www.premierguardusa.com](http://www.premierguardusa.com)



Do Not Reuse



Does Not  
Contain  
Natural Rubber  
Latex



Do Not Use  
if Package is  
Damaged



Protect from  
sunlight



Protect from  
moisture

PREMIER | GUARD  
USA

